


14. Storing Data in a Database

- Select the Database option. **1**
- If required, select a database from the drop-down. **2**
- Click Store Volume. **3**

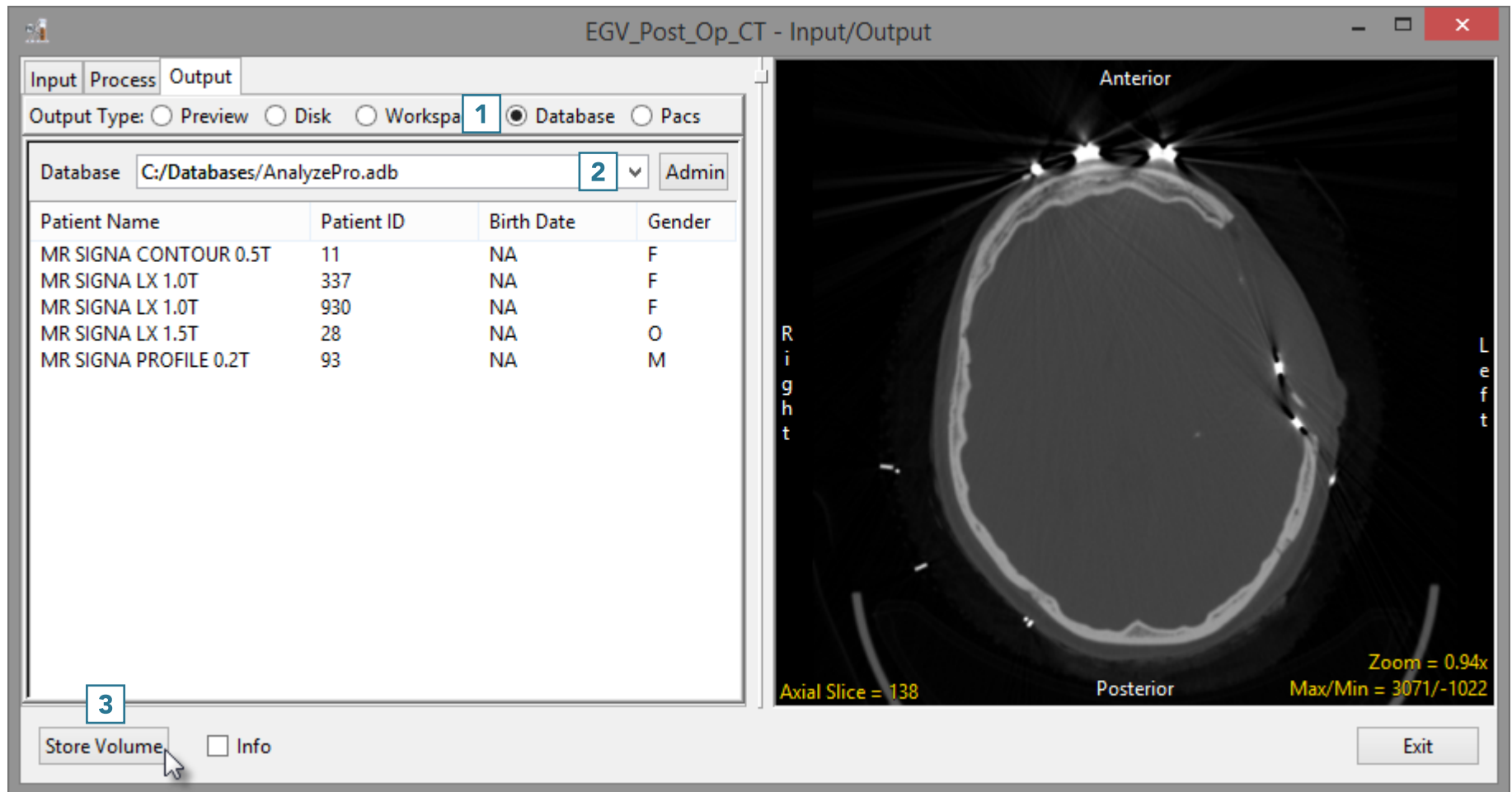

- The Edit/Add DICOM tags window will open. Use this tool to add or edit any DICOM tag information.
- Click Apply Tags and Continue. 4
- The data will be exported to and merged into the database.

Edit DICOM tags

Patient

ID (0010,0020)	500
Name (0010,0010)	Joe Bloggs
DOB (0010,0030)	
Gender (0010,0040)	M

Study

ID (0020,0010)	
Description (0008,1030)	
Date (0008,0020)	
Accession (0008,0050)	
Modality (0008,0060)	
Portion (0008,0015)	

(0020,000d) Make New Study Instance UID

Series

ID (0020,0011)	
Orientation	<input checked="" type="radio"/> Axial <input type="radio"/> Coronal <input type="radio"/> Sagittal
Description (0008,103e)	
Procname (0008,1030)	

(0020,000e) Make New Series Instance UID

Other Tags

(gggg,eeee)	
-------------	--

4
Apply Tags and Continue
Clear
Add Tag
Cancel